

The FlexorAide+TM/Patent Pending


3 Step Instruction Sheet

The purpose of The FlexorAide+ is to provide safe, immediate immobilization to prevent further injury, and to promote continued support in the healing process in conjunction with your veterinarian, regarding the appropriate manipulation of the limb and proper setting of the splint during the stages of the progressive healing process.




Scan the QR Code
for Video
Instructions

1) Detach the hook and loop and loosen lower wing nuts and carousel nuts. (It is not necessary to remove fastening nuts unless size adjustment is necessary)

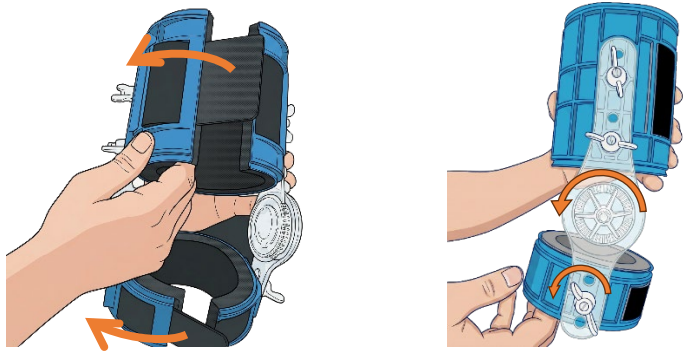


2) Apply the FlexorAide+ over the front of the affected limb.

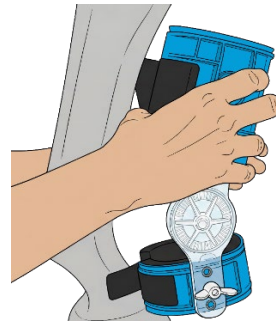

3) Secure the hook and loop and tighten all fastening nuts. (Snug tight only)

***Stop tightening at point of resistance to prevent damaging any nuts/bolts**

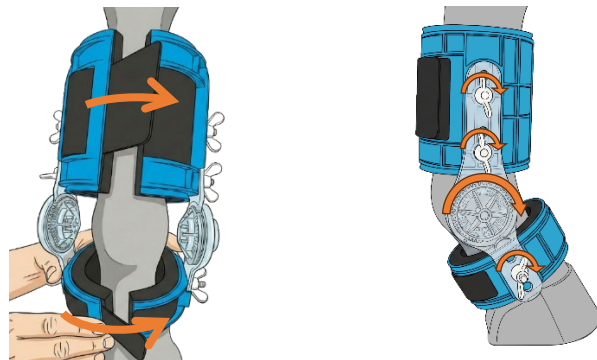

Please note: The following step and ongoing healing process should proceed **ONLY** in conjunction with your veterinarian to optimize chances of full recovery.

When ready to position the limb, loosen geared hinges and **ONLY** the lower hinge arms to allow manipulation of the limb to desired position. **(The FlexorAide+ does not need to be removed to manipulate limb.)**

Once in place, tighten lower hinge arms **FIRST** (to ensure alignment) then geared hinges.

Only the center geared hinges and lower hinge arms will be repeatedly loosened and tightened for appropriate positioning during the progressive healing process.